POWERCHARGE **CLASSIC**

Version 2025:1
© Swede Electronics AB

Bruksanvisning (SE)	5
Bruksanvisning (NO)	8
Betjeningsvejledning (DA)	11
Käyttöohje (FI)	14
User manual (UK)	17
Bedienungsanleitung (DE)	20
Gebruiksaanwijzing (NL)	23
Guide d'utilisation (FR)	26
Manuale d'istruzioni (IT)	29
ユーザーマニュアル(JA)	32

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info@swedeelec.se • www.swedeelec.se

IMPORTANT SAFETY INSTRUCTIONS

This manual contains important safety and operating instructions for your battery charger. Before charging, read the instructions. Keep the manual for future reference.

This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved.

Children shall not be permitted to play with the appliance.

Cleaning and user maintenance shall not be undertaken by children without supervision.

INSTRUCTIONS FOR USE

- 1** Plug the power cord of the battery charger into the socket.

- 2** Connect the charging plug to the battery¹.

¹ See the manual of the vehicle for location of the charging socket. The power cord and charging plug can be connected in any order.

- 3** The yellow light flashes during the automatic charging process.
The charger is automatic and adjusts and limits the current and voltage depending of the battery status.

- 4** The yellow light is constantly lit when the charging is completed, and the battery is ready for use and during maintenance charging.
As the charger uses very little power and cannot overcharge the battery, it can be left connected to the battery until the vehicle is required for use.

- 5** Unplug the charging plug from the battery² before using the vehicle.

² See the manual of the vehicle for location of the charging socket.

- 6** The power cord may be unplugged.

WARNING

- Batteries emit explosive gases during charging. Prevent flames and sparks.
- Provide adequate ventilation during charging.
- The charger is intended only for lead batteries with 12 cells (24V).
- Never charge a frozen battery.
- Never charge non-rechargeable batteries.
- Medical electrical equipment requires special safety precautions and may only be installed and used according to the manufacturer's instructions.
- Portable and mobile radio communications equipment can affect medical devices.

INDICATIONS

INDICATION	SIGNIFICANCE / CAUSE	ACTION
 Steady green light	= Connected to power.	
 Flashing yellow light	= Charging in progress.	
 Steady yellow light	= Charging completed. Trickle charging.	The battery charger should stay plugged in until the vehicle is used.
 Steady red light	= Battery cables are connected to the wrong poles, i.e. reverse polarity or the charger is overheated or battery fault.	Please contact your service workshop.

In case faults cannot be corrected, please contact an authorised service workshop.

CARE AND MAINTENANCE

- Prior to each charge, check that the cables and plugs are not damaged or worn. If this is the case, the cable/ plug must be replaced immediately by the manufacturer or by an authorised service workshop.
- For the best performance, economics, and service life of the charger, battery, and optimum driving distance per charging cycle, the following advice should be followed:
 - The charger, plugs, and batteries must be kept clean from dirt, dust, and oxidation.
 - Switch off the vehicle when it is not in use.
 - Charge the battery daily, or as soon as possible when it is empty.
 - Use the charger to maintenance charge the battery when the vehicle is in storage.
- If one of the plugs becomes hot during operation, this can be an indication that the plug is worn or damaged. In such cases both the male and female plugs should be replaced.
Cables and plugs must be changed only by the manufacturer or by an authorised service workshop
- The charger can be cleaned as required using a damp cloth.

TECHNICAL SPECIFICATIONS

TYPE NO	POWERCHARGE CLASSIC 8 A	POWERCHARGE CLASSIC 10 A	POWERCHARGE CLASSIC 12 A
Rated current/voltage	8A  24V	10A  24V	12A  24V
Battery type/voltage/size	AGM, GEL 24V, 12 cells 20 to 80Ah* (C5)	AGM, GEL 24V, 12 cells 30 to 100Ah* (C5)	AGM, GEL 24V, 12 cells 50 to 120Ah* (C5)
Mains voltage	230V 50Hz	115V 60Hz/230V 50Hz	115V 60Hz/230V 50Hz
Max power consumption	255W	325W	390W
Temperature range	5°C–25°C	5°C–25°C	5°C–25°C
Weight	1.8 kg	2.4 kg	2.8 kg
Measurements (HxWxD) incl. handle	158x189x62 mm	215x189x62 mm	215x189x90 mm
Casing class	IP54	IP54	IP54
Insulation class	II (double insulated)	II (double insulated)	II (double insulated)
Safety class	Type B	Type B	Type B

* Powercharge Classic is capable of charging batteries up to 80 % of the maximum battery capacity in less than 8 hours.

Standard pin assignment: 1 = +, 2 = –, 3 = –

POWERCHARGE CLASSIC FEATURES PROTECTION FOR:

- Reversed polarity
- Shortening of cables
- Sparks when connecting to power or battery
- Overheating

The battery charger is equipped with thermal protection, but gets warm during charging.

Do not expose the battery charger to direct sun light.

Do not lower the battery charger into water or cover it with snow, whether completely or partly.

Cables and plugs may only be replaced by the manufacturer or an authorized service workshop.

Only the casing is IP54

Only for indoor use

Read the manual before use

CE-MARKINGS:

The manufacturer hereby declares that the Powercharge Classic charger complies with the following standards:

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Swede Electronics AB, Fabriksvägen 8, SE-786 33 Vansbro, Sweden